







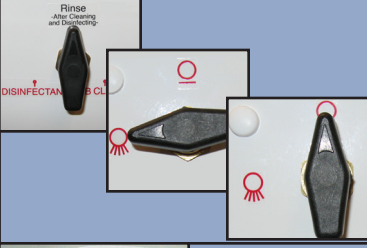



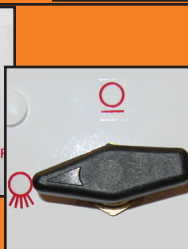


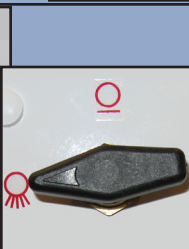


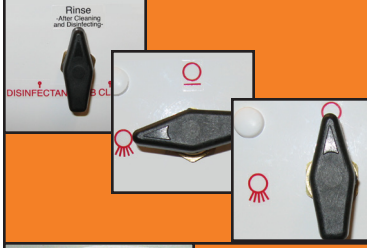



### Bath Cleaning Process

### Bath Disinfecting Process

	Place the chair in the tub, release the carrier from the tub, and close the door.
	Close the Tub Drain.
	Turn the Blower on.
	Turn the Selector Knob to "TUB CLEANER" and the Control Knob to "On."
	When there is about 2 inches of cleaning solution in the foot well; turn the Control Knob to "Off."
	Turn the Blower off.
	Lift seat bottom off chair. Use the cleaning solution to scrub the tub, chair, and underneath seat bottom.
	Open the Tub Drain and turn the Blower on.
	Turn the Selector Knob to "Rinse" and the Control Knob to "On." When clear water comes out of all outlets, turn the Control Knob to "Off."
	Leave the Blower on for 60 seconds to force the remaining water out of the air lines. Turn the Blower off. Use the shower wand to rinse the tub and chair.

	Place the chair in the tub, release the carrier from the tub, and close the door.
	Close the Tub Drain.
	Turn the Blower on.
	Turn the Selector Knob to "DISINFECTANT" and the Control Knob to "On."
	When there is about 2 inches of disinfectant solution in the foot well; turn the Control Knob to "Off."
	Turn the Blower off.
	Lift seat bottom off chair. Use the disinfecting solution to scrub the tub, chair, and underneath seat bottom.
	Leave wet for 10 minutes. After 10 minutes, open the Tub Drain and turn the Blower on.
	Turn the Selector Knob to "Rinse" and the Control Knob to "On." When clear water comes out of all outlets, turn the Control Knob to "Off."
	Leave the Blower on for 60 seconds to force the remaining water out of the air lines. Turn the Blower off. Use the shower wand to rinse the tub and chair.