

APOLLO MOBILE DIGITAL SCALE OPERATION

Before operating check to see that:

- A. The chair is locked into place on the carrier.
- B. The carrier is disconnected from the tub.
- C. Make sure scale carrier is on a hard level surface.
- D. Make sure that you are only weighing the resident - remove towels, robes, shoes, etc.

OPERATION:

1. To turn the scale on press the “ON/ZERO” pad. The display will go through a startup and settle on some weight or 0.0. If the display shows any number other than 0.0, press the **ON/ZERO** pad. The display will flash **ZERO** and go to 0.0. The scale is now ready for weighing.
2. Place the individual onto the stretcher/chair and lift them clear of contact with any fixed objects, i.e., bed or chair. This will ensure proper weighing.
3. Read the weight and record. If the display goes off before you read the weight, press the “ON/ZERO” pad one time to start the scale again and weight will be shown.
4. Be sure patient is not pushing on the floor with their feet.
5. Record the weight shown on the digital readout.

**NOTE: THE SCALE WILL SHUT OFF
AUTOMATICALLY AFTER 2 MINUTES.**

**Be sure the Residents feet are
not on the foot rests. For the
most accurate weights, be
sure the foot rests are down
and the foot pads are flipped
up and out of the way.**

SEE ILLUSTRATION

