

APOLLO SCALE CALIBRATION

1. PRESS THE **ON/ZERO** BUTTON.
2. WHEN SCALE HAS TURNED ON PRESS THE **ON/ZERO** BUTTON AGAIN TO ZERO THE SCALE.
3. NOW AGAIN PRESS THE **ON/ZERO** BUTTON **AND** THEN THE HIDDEN BUTTON BENEATH THE LAST “O” IN APOLLO. HOLD THEM **BOTH** DOWN UNTIL “**CAL**” APPEARS.
4. THE DISPLAY WILL NOW SHOW “**C 0**” AT THIS POINT AGAIN PRESS THE **ON/ZERO** BUTTON. THE SCALE WILL NOW GO TO “**16**” OR “**8**” AND BEGIN TO COUNT DOWN TO ONE.
5. ONCE THE SCALE REACHES ONE IT WILL DISPLAY “**C 25**”.
6. IT IS NOW ASKING YOU TO SHOW IT WHAT 25LBS IS. NOW YOU PLACE YOUR **25LB WEIGHT** ON THE CHAIR.
7. NOW THAT YOU HAVE 25LBS ON THE CHAIR AGAIN PRESS THE **ON/ZERO** BUTTON.
8. THE SCALE WILL ONCE AGAIN GO TO “**16**” OR “**8**” AND COUNT DOWN TO ONE AND THEN SHOW 25LBS.
9. THE SCALE NOW WILL RECOGNIZE THAT WEIGHT AS **25LBS**.
10. REMOVE THE WEIGHT AND THE SCALE SHOULD RETURN TO ZERO.

NOTE: IT IS EXTREMELY IMPORTANT THAT THE WEIGHT YOU USE TO CALIBRATE YOUR SCALE IS EXACTLY 25LBS. IF NOT THE MORE OUNCES YOU ARE OFF EITHER TO THE HEAVY OR LIGHT SIDE WILL AFFECT YOUR RESIDENTS WEIGHTS.

ALSO NOTE THAT IN STEP 3 IF CONV APPEARS, IT USUALLY MEANS THAT YOU PRESSED THE HIDDEN BUTTON BEHIND THE LAST “O” BEFORE YOU PRESSED THE ON/ZERO BUTTON. THAT BUTTON CONVERTS THE SCALE BETWEEN LBS AND KG WHEN PRESSED ALL BY ITS SELF.

THIS SCALE ALSO HAS A LAST WEIGHT RECALL FUNCTION. THIS MEANS IF YOU SHOULD FORGET TO WRITE DOWN YOUR LAST WEIGHT. BY PRESSING THE FIRST “O” IN APOLLO THE LAST WEIGHT WILL BE DISPLAYED FOR YOU.

Note: This scale is not waterproof - Not for showering