















Bath Cleaning Process

Bath Disinfecting Process

	<p>Close the door.</p> <p>Close the Tub Drain.</p>
	<p>Turn the Selector Knob to “TUB CLEANER” and the Control Knob to “On.”</p>
	<p>Turn the whirlpool on.</p> <p>When there is about 2 inches of cleaning solution in the foot well, turn the whirlpool off.</p>
	<p>Turn the Control Knob to “Off.”</p> <p>Use the cleaning solution to scrub the tub.</p>
	<p>Open the Tub Drain.</p> <p>Turn the Selector Knob to “Rinse” and the Control Knob to “On.”</p>
	<p>Turn the whirlpool on.</p> <p>When clear water comes out of all the jets, turn the whirlpool off.</p>
	<p>Turn the Control Knob to “Off.”</p> <p>Use the shower wand to rinse the tub.</p>

	<p>Close the door.</p> <p>Close the Tub Drain.</p>
	<p>Turn the Selector Knob to “DISINFECTANT” and the Control Knob to “On.”</p>
	<p>Turn the whirlpool on.</p> <p>When there is about 2 inches of disinfectant solution in the foot well, turn the whirlpool off.</p>
	<p>Turn the Control Knob to “Off.”</p> <p>Use the disinfecting solution to scrub the tub. Leave wet for 10 minutes.</p>
	<p>After 10 minutes, open the tub drain.</p> <p>Turn the Selector Knob to “Rinse” and the Control Knob to “On.”</p>
	<p>Turn the whirlpool on.</p> <p>When clear water comes out of all the jets, turn the whirlpool off.</p>
	<p>Turn the Control Knob to “Off.”</p> <p>Use the shower wand to rinse the tub.</p>